STRATOGEL

HOJA DE INSTRUCCIONES	ES	2
WORKING INSTRUCTIONS	EN	4
NOTICE D'EMPLOI	FR	6
ISTRUZIONI PER L'USO	IT	8
GEBRAUCHSANWEISUNG	DE	10

STRATOGEL

DESCRIPCIÓN

GEL SEPARADOR PARA MODELOS 3D.

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DESCRIPTION

SEPARATING GEL FOR 3D MODELS.

Stratogel is designed to separate an orthodontic acrylic resin such as TECHNO SIN or TECHNO SIN Tixo, from a model printed with 3D resin. The correct separation with other types of resins is not guaranteed, in this case tests must be carried out.

This is a medium viscosity gel that is applied to the surface of the 3D resin model with a brush and that dries in air.

Even though the gel is thick, once applied to the surface it spreads by sliding and creating a thin film which, once dry, creates a thin plastic layer that acts as a separating layer between the 3D model and the acrylic resin.

It is supplied with a suitable viscosity to be effective with only one coat applied. With the first layer the thickness of the film created (approximately 12 microns thick) is already effective for most 3D printing surfaces, however in case of very rough surfaces (prints of more than 100 microns) a second layer can be applied on top of the first one, always allowing the first one to dry beforehand.

INSTRUCTIONS OF USE

Apply a few drops on the model and spread it with the brush. Dry at low pressure with compressed air for 30 sec.

For a fast drying (30 seconds) it is recommended to dry the gel with compressed air at low pressure, otherwise the air drying is slow (approx. 30 min.).

As it is not a greasing agent, but rather creates a plastic film, once the acrylic resin has polymerized, in order to separate it, we need to introduce the tip of a blade between the acrylic resin and the 3D model in order to lever it a little and get the acrylic resin to detach from the model.

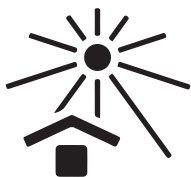
Once separated, the plastic film remains adhered to the acrylic resin, leaving the 3D model clean. This plastic film can be easily removed by washing with water and a brush. It leaves no residue, does not attack the acrylic resin or the 3D model, does not alter its properties and does not contain any toxic components.

ADVANTAGES

- Easy application: a single coat is applied with a brush
- Creates a very thin separating layer
- Does not contaminate 3D resin or acrylics
- Easily removed with water

EXPIRY

The stability of the product is 3 years, respecting the storage conditions. (Note: The expiry date is considered with the product in his original packing and before been opened).

STORAGE

Keep the product tightly closed in its original package and in a cool place protected from sunlight. Close the bottle after each use to avoid evaporation.

SAFETY

- H225 Highly flammable liquid and vapor
- H319 Causes serious eye irritation
- H336 May cause drowsiness or dizziness
- P210 Keep away from heat, sparks, open flames and hot surfaces. No smoking.
- P261 Avoid breathing mist/vapors/spray.
- P303+P361+P353 IF ON SKIN (or hair): Remove immediately all contaminated clothing. Rinse skin with water/shower.
- P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
- P304+P340 IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing.
- P403+P233+P235 Store in a well-ventilated place. Keep container tightly closed. Keep cool.
- P501 Dispose of contents and container in accordance with local, regional and national regulations.

PACKAGING

3280-100	Bouteille 100 ml	(3.5 fl oz)
3280-M10	Test pack 25 ml	(0.8 fl oz)

GUARANTEE

Our recommendations referring to the techniques applied, whether verbally, in writing or in the form of practical instructions, are based on our own experiences and/or tests, and must therefore be considered only as instructions.

For further information, see the safety data sheet of STRATOGEL 3D. Order them to your local dealer or visit our website to download them.

IMPORTANT: Check our website for the most updated Instruction Manual version, as this information is subject to continuous review.

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